



Libra+ Audio Transmitter (212-637)

Introduction

The Libra+ Audio Transmitter is an adaptation of the Libra+ Doorbell push converter. The converter allows connections to existing doorbells or pushes. This adaptation has added a sound sensor similar to those found in other doorbell/telephone bell alerting products.

You can now monitor, without the need for long connecting leads or where there is no mains power available, any loud sounds. These alerts can then be sent to any receiver in the Libra+ range. For example, you can now monitor a door entry phone, telephone bell or other door bell where only an audio connection is possible.

Instructions for use

Insert the coin cell battery into the bell push and program the bell push as per the Libra+ instruction booklet.

The main push button of the bell push can subsequently be used to test the transmitter.

Attach the Libra+ bell push to a convenient surface near to the telephone or doorbell etc.

Locate the position on the bell where the loudest bell sound comes from.

Simply attach the sound sensor with its holes facing the speaker of the telephone etc. to the body of the telephone using either the sticky tape already attached to the sensor or with the Velcro roundels.

When using the Velcro roundels please ensure the centre hole is clear and does not obstruct the sensor holes.

Using the Velcro roundels will allow the sensor to be removed for cleaning purposes.

Test the system by activating the phone or door bell.

If you find that the system is activated by normal household sounds such as general conversation or TV sounds you can turn down the sensitivity of this transmitter by removing the rubber cover on the rear of the item, and adjusting the small trimmer in an anti clockwise direction until it is no longer activated by general sounds.

Battery life - Up to 1 year's use can be expected when used in conjunction with the audio sensor. Battery life is about half that could be expected with normal usage of this bell push.

Sarabec Ltd
15 High Force Road
Middlesbrough
TS2 1RH

Telephone (01642 247789)
Fax (01642 230827)
Minicom (01642 251310)
Email: sales@sarabec.co.uk
Web: www.sarabec.com



is a mark of Sarabec Ltd.