HUMANTECHNIK

**VIBtimer**

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**GB** Operating Instructions Page 6

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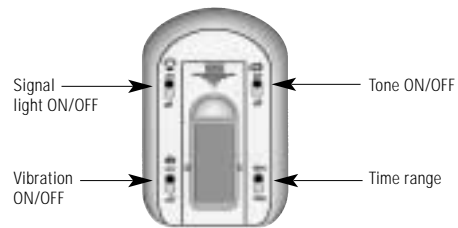
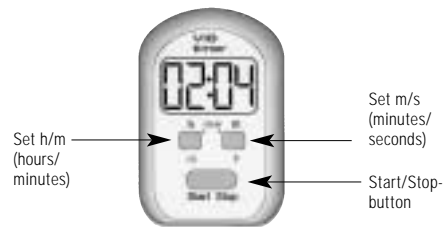
**Time range**

Setting h/m: maximum countdown time: 19 hours 59 minutes  
 Setting m/s: maximum countdown time: 99 minutes 59 seconds

Congratulations on purchasing the VIBtimer. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully.

**Setting the count down timer:**

Set the display to 0:00 by pressing the „h/m“ and „m/s“ buttons at the same time.

Set the count down time by pressing the buttons „h/m“ and „m/s“.

Start the count down by pressing the „Start/Stop“ button once. (Pressing the „Start/ Stop“ button once again interrupts the count down.)

The signal is activated after the set time is reached. (Signalling light and/or vibration and/or acoustic signal).

**Setting the count up timer:**

Set the display to 0:00 by pressing the „h/m“ and „m/s“ buttons at the same time

Start the count up by pressing the „Start/Stop“ button once. (Pressing the „Start/ Stop“ button once again interrupts the count up.)

***No signal can be activated in this operating mode!***

**Battery change**

The VIBtimer is operated with 2 alkali batteries (type AAA). Open the battery compartment by pushing the cover into the direction of the arrow (see drawing). Insert 2 batteries according to the marking in the battery compartment.

**Maintenance and care**

The unit is maintenance free. To clean, wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

**Warranty**

The VIBtimer shows a high operational reliability. Please contact your local dealer if any dysfunctions occur despite correct handling.

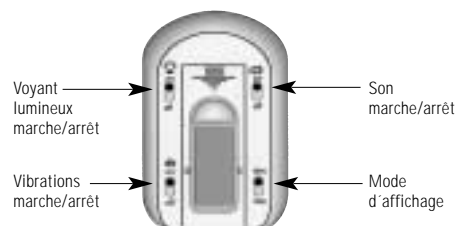
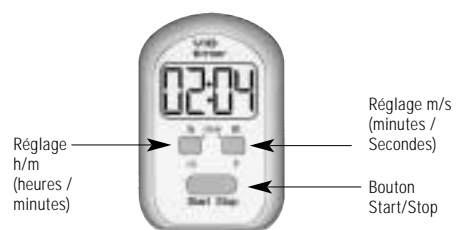
This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

**Technical data**

Measurements:	72x48x30 mm (HxWxD)
Colour:	silver
Weight:	68 g
Power supply:	2 x 1,5 V micro alkali batteries (type AAA)

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**Humantechnik GHL AG**
ELECTRONIQUE S.A.R.L**